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[ENG] **DROP defence®** is an anti UV ophthalmic protective solution based on Riboflavin sodium phosphate 0,05%, Vitamin E TPGS, MSM, amino acids and Hyaluronic Acid. It acts as a shield for the ocular surface against UV and blue light, and improves ocular surface lubrication, contributing to corneal epithelium protection, against different stress sources. Sodium Hyaluronate mechanically interacts with tear film mucin layer, retaining water and promoting longer stability of the ocular surface. Amino acids' synergic action has a nourishing function for the ocular surface, other than pH and tear osmolality regulation, improving epithelial protection against stress caused by UV and blue light. Vitamin E TPGS promotes ocular surface wettability. The ophthalmic solution has a shielding, protective and anti-oxidant action against UV damage, and accelerates the natural repairing process. **DROP defence®** is a unique and certified Personal Protective Equipment that protects the cornea, crystalline and retina against UV radiation, blue light, artificial light sources and prolonged sun exposure, preventing damages like photo-keratitis, photo-keratokonjunktivitis, cataract and AMD. **DROP defence®** keeps the ocular surface hydrated and protected. **DROP defence®** can be used with wearable contact lenses. **DROP defence®** is a Personal Protective Equipment. **INDICATIONS:** **DROP defence®** has a protective action against UV, blue light, sunlight and artificial light harmful effects. **DROP defence®** prevents photo-keratitis and photo-keratokonjunktivitis occurrence, due to UV exposure. **DROP defence®** prevents cataract occurrence, due to UV exposure. **DROP defence®** prevents macular ageing, caused by UV and blue light exposure. **DROP defence®** is indicated in case of long exposure to computers, video terminals and television. **DROP defence®** is indicated in case of intense and prolonged UV rays exposure. **DROP defence®** is indicated for outdoor sports and activities. **DROP defence®** is indicated for exposures to high UV environments (mountain, sea, tropical areas, etc...). Its application does not affect visual acuity or chromatic perception and keeps contrast sensibility unchanged. **INSTRUCTIONS FOR USE:** 1. Open the dispenser and remove the protection cap. 2. Gently push the dispenser and use one or two drops in each eye. Repeat this step if needed. Close the dispenser after use: the remaining product can be used up to 2 months after the first opening. The product can be used with wearable contact lenses.

COMPOSITION: Riboflavin sodium phosphate 0,05%, Sodium chloride, Vitamin E TPGS, Sodium phosphate bibasic dodecahydrate, Sodium hyaluronate, Methyl Sulfonyl Methane, Sodium phosphate monobasic monohydrate, L-proline, L-glycine, L-leucine, L-lysine chlorhydrate, Purified water. **WARNINGS AND PRECAUTIONS:** - The product is intended for external ophthalmic use only. The product is intended for single patient use. Do not use in cases of known intolerance or hypersensitivity to its components. During product application, do not touch the eye or any other surface with the dispenser tip. Do not use in case of damaged or unclosed package before the first opening. In case of problems during use, discontinue treatment and consult a doctor. Do not use after the expiry date indicated on the package. The expiry date refers to unopened and correctly stored product. Before use, carefully read the instructions found on the leaflet. In rare cases, phenomena of corneal microcalcifications have been found in patients with previous corneal pathologies and treated with ophthalmic preparations containing phosphates. In this case, it is recommended to use the product under the supervision of the doctor. Do not look directly at hazardous light sources such as the sun, laser, welding machine, etc. Keep away from children. Do not waste into the environment after use. It is possible to report adverse events to the manufacturer via the email address info@iromedgroup.com. By reporting adverse events, you can contribute to the safety of this product. Any severe events should be notified to the manufacturer and the competent authority of the country in which this product has been used. **PACKAGING:** One 10 ml multidose dispenser, preservative free. **STORAGE:** Immediately close the dispenser after use; the remaining product can be used up to 2 months after the first opening. Store at a temperature lower than 30°C. **EXPIRY** With an undamaged package, do not use after the expiry date indicated on both the box and the dispenser. Do not separate the dispenser from the box and the leaflet, to keep all the information for a correct use in the same place.

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0373 MEDICAL DEVICE compliant to directive 93/42/CEE and its amendments

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PERSONAL PROTECTIVE EQUIPMENT of category I, compliant to directive 89/686/ CEE

Po uporabi ne zavrzite v okolje /
Do not waste into the
environment after use

Hranite zunaj dosega
otrok / Keep away from
children

Rev. izd. 02, 02, 6/2021

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IROMED GROUP S.r.l.
SERVImed Industrial S.p.A.